

SMART VISUAL EAR CLEANER

Visual Ear Pick

Instruction Manual



NOTE:


For a better experience, please read this manual carefully before use and keep it for future reference.

3. PRODUCT SPECIFICATIONS

Product Name:	Smart Visual Ear Cleaner
Model:	Y24
Camera Diameter:	3.5mm
Pixel:	1080P
Input:	5V=1A
Battery:	3.7V 320mAh
Charging Time:	About 1.5 Hours
Working Time:	About 60 Minutes
WiFi Frequency:	2.4GHz
Operating Temperature:	-10°C ~ 45°C
Storage Temperature:	-10°C ~ 60°C

1. PRODUCT INTRODUCTION

This product is a smart visual ear cleaner with a high-definition camera and LED light. It connects to your smartphone via WiFi, allowing you to see inside the ear canal in real time and clean it safely and effectively.

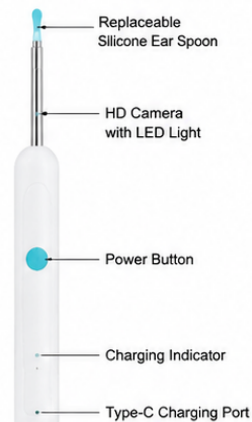

Model: Y24
Input: 5V=1A
Battery: 3.7V
Made in China

Storage Box

4. SAFETY PRECAUTIONS

- This product is for ear cleaning only, not for medical use.
- Children under 12 should use under adult supervision.
- Do not use if you have ear infections, ear injuries or had ear surgery.
- Do not insert the ear spoon too deep into the ear canal.
- If you feel discomfort during use, please stop immediately.
- Keep the device dry and away from fire, high temperature and corrosive liquids.
- Do not disassemble the device.

5. CLEANING AND MAINTENANCE

- After each use, remove the silicone ear spoon and wash it with clean water or alcohol.
- Wipe the lens and body with a soft, dry cloth.
- Put the device and accessories back into the storage box and keep it dry.

2. HOW TO USE



1

Download APP

Search for "Yscope" in App Store or Google Play, download and install it.



2

Connect WiFi

Long press the power button for 2s to turn on the device. Open the APP and follow the instructions to connect the WiFi: Y24-XXXX.



3

Start Using

After connection, you can see real-time video on your phone. Gently insert the ear spoon into the ear canal and clean carefully.



4

Replace Ear Spoon

Gently pull out the silicone ear spoon and replace it with a new one.

6. PACKAGE CONTENTS

