




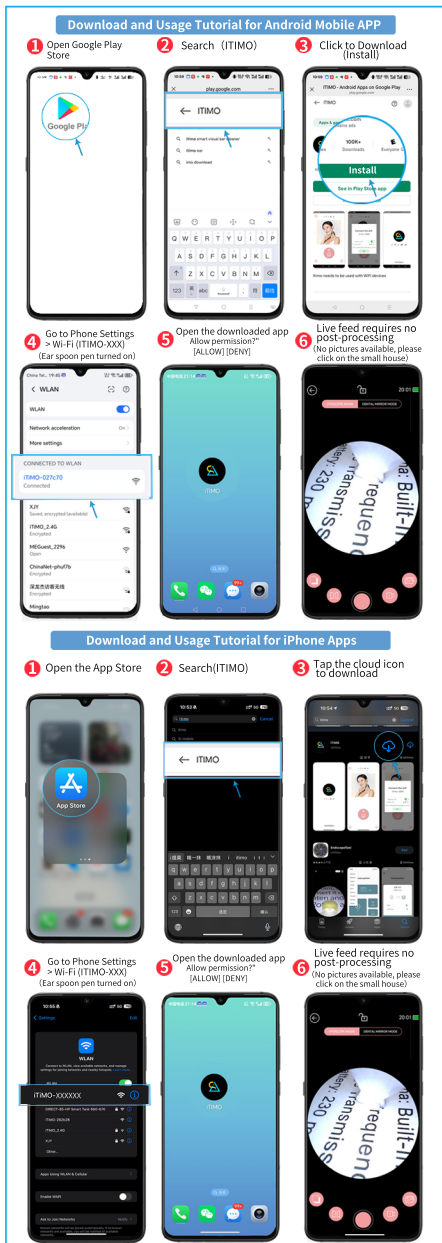

itimo

App Store下载 >

Android(China)下载 >

Google Play下载 >

Spare download QR code

Friendly Reminder:
 This is completely free software. Please note:
 If you encounter any ads, ignore them (do not interact).
 The software will never charge you any fees.
 Any accidental charges due to user error are unrelated to this software. For safety, please download only from official sources.

Specification

Product Name: Visual Earwax Removal Endoscope
 Network Protocol: IEEE 802.11b/g/n, 2.4GHz
 Antenna: Built-in IPEX antenna
 Working Frequency: 2.4GHZ
 Resolution: 1920 * 1080P
 Image Transmission Rate: 30fps
 Battery: 130 mAh lithium battery
 Working Time: About 35 minutes
 Charging Time: 30 minutes
 Input: 5V 0.3A

Common Problems and Solutions

1. When there is no image or the image is still, please make sure that the mobile phone and the otoscope are connected with wifi, and you can try to restart the otoscope Or APP.
2. If you cannot turn on or the LED lights flicker during use, please charge the product in time to ensure that the battery is charged.
3. If it prompts that it cannot be accessed when connecting, it may be that the otoscope is connected to another connected mobile phone. Restart the otoscope and reconnect.
4. When the product is turned on, the indicator light flashes, waiting for connection, after connection, the indicator light keeps on, when the product is low battery, the indicator light flashes quickly, when the product is charging, the indicator light stays on, and then go off when it is full.
5. Since the ear canal is curved, please adjust the direction of the otoscope slightly when using it until you can see the eardrum clearly.
6. The focal length of this product is fixed, and the image of the object is clearest at a distance of 1.5-2 cm, Please adjust the distance, until the image is clear.

WARNING

1. Please observe your surroundings before use. do not use the product in an environment where people are running. to avoid injury.
2. This product is not suitable for children under 3 years old.
3. Do not allow children to use this product alone to avoid accidental injury.
4. Do not put the product in liquid to avoid damage
5. The product has a built-in rechargeable lithium battery. If it is not used frequently, it must be charged once a month.
6. When using, the product temperature will rise slightly (maximum not more than about 45 °C), please rest assured to use.
7. The products is forbidden to use a charger or mobile power supply with a voltage higher than 5v to charge.
8. Accessories can not be replaced to the host, suitable for use alone.

(When the product is charging, the red light turns on and goes off when fully charged. The blue light serves as the signal light.)

正面

反面