

HAIR REMOVAL / SAFE & NON-INVASIVE / MOLE & SPOT REMOVAL

Xiao Bai 810 Hair Removal Device

instruction manual



Product list

Introduction

Principle of Instrument Operation	01
Instrument parameters	01
Instrument appearance	02

Instructions for use

Interface operation instructions	04
Parameter setting table	06
Taboo group.....	06
Matters needing attention.....	07
Instrument parts list	07

Introduction

Principle of Instrument Operation

According to the principle of selective photothermodynamics, through reasonable adjustment of wavelength, energy and pulse width, light energy can pass through the skin surface to the hair root hair follicle, so that the hair follicle tissue is damaged, slow growth or no growth. The rapid freezing point hair removal system adopts a unique freezing technology to cool the local epidermis to 5 degrees Celsius, overcoming the shortcomings of cold touch lighting, convenient and simple operation. No damage to the skin, easy hair removal.

Instrument parameters

Hair removal instrument parameters	
Facula	12*26mm
Pixel	12寸
The number of operations	50000000发
Wave spread	1064/808/755
Frequency	1-10Hz
Energy	1-50J/cm ²
Overall power	2000W
Pulse width	1MS-20MS
Voltage	200-240V
Heat dissipation	Semiconductor refrigeration + air cooling + TEC refrigeration
Package size	61.5cm*55cm*57cm
Language	Supports various language customization
Hair removal instrument parameters	

Introduction

Instrument appearance



Instrument Front

(The picture is subject to object)

- ① Super clear LCD screen
- ② hair removal handle
- ③ On/Off

Introduction

Instrument appearance




Instrument Back

(The picture is subject to object)

- | | | |
|-----------------------|----------------|----------------|
| ① hair removal socket | ② Power switch | ③ Power socket |
| ④ Water inlet | ⑤ Gas vent | ⑥ Drain hole |

Instructions for use

Interface description

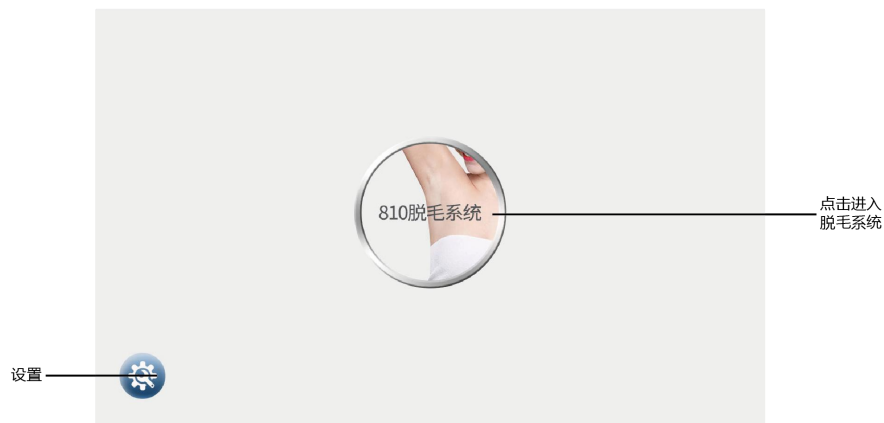

(1-1)

Open the instrument and the main interface appears. Touch the screen to enter the handle selection mode (for example, select 810 mode).



(1-2)

Go to (1-2), select the gender, and then proceed to the next page




(1-3)

Enter the operation interface diagram (1-3), select the body part, select the skin color, and click to enter the next page


Instructions for use

Interface description

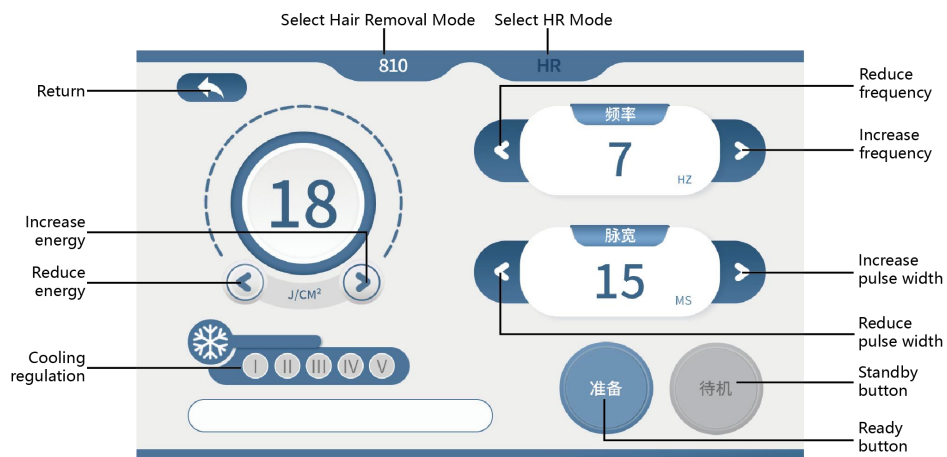

(1-4)

Enter the operation interface "Figure (1-3), there are 810 hair removal and rejuvenation mode (SR), adjust the energy, adjust from 20, adjust the frequency, adjust from 3-4, pulse width from 10-12, adjust the energy size according to the actual situation of customer operation, click "Ready "to start the operation.

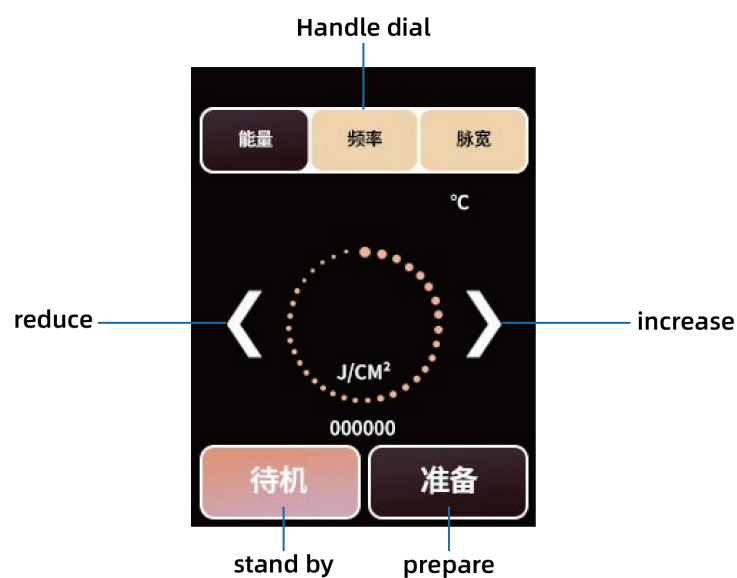

(1-5)

At the same time, the energy, frequency and pulse width can be adjusted on the handle

Instructions for use

Parameter setting table

Hair color	Hair color	Pulse width	Frequency	Energy	Time
Yellow skin	Brown	10~15	3~6	Start at 20	10 to 15 minutes
Fair skin	Golden	10~15	3~6	Start at 25	10 to 15 minutes
Black and yellow bark	Black	10~15	3~6	Start at 15	10 to 15 minutes

Taboo group

1. Pregnant women, high blood pressure, heart disease, diabetes;
2. People who are allergic to light are prohibited;
3. People with open wounds;
4. People who cannot avoid private life in the near future;
5. Prohibited during menstruation.

Instructions for use

Matters need attention

- 1, do not use soap, shower gel within 6 hours of hair removal, and do not make up the treatment area. Isowash treatment,Area, do not use cosmetics on the day do not rub the treatment area.
2. Only use mild skin care formulas after freezing hair removal.
3. Dress loosely and keep dry after freezing hair removal.
4. do not expose to the sun after freezing hair removal, sauna, swimming.
5. do not use scrub, alcohol, antiperspirant after freezing hair removal, and avoid heat in the treatment area when taking a bath,Water, should use warm water.
6. within a week do not eat photosensitive food (such as celery, spinach, vegetables, lemons, etc.) and photosensitive drugs (such as tetracycline, etc.), spicy and irritating food.

Instrument parts list

Part	Quantity
Hair removal handle	1
Power cord	1
Carbon dust	1
Protective spectacles	1

Attention:

The machine is a photoelectric product,During operation, Be careful not to aim at people's eyes, after operation, you need to put the handle into the protective case, revent injury to people by misoperation.